




ISO 13485

# Ufit<sup>®</sup>

## dental implant system

Submerged Fixture  
Submerged Abutment  
Multi-unit Autment  
Surgical tools

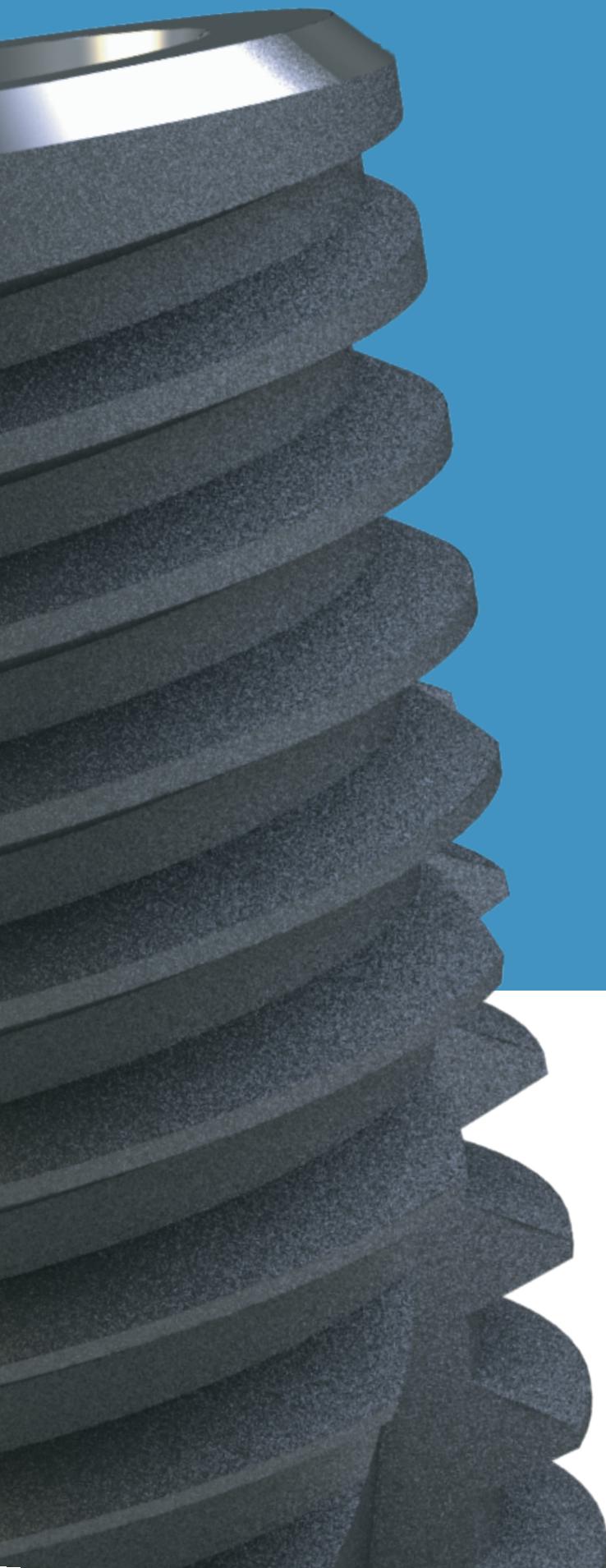



# Contents



## Contents.

Company History	04-05
Fixtures	06
SYLBUTMENT	07-11
Fixture Features	12-13
Flow Charts	14-15
Gt2 Fixture / Nt2 Fixture	16-17
Abutments	18-31
Multi-Unit Abutment	32-35
Surgical Tools	36-40
Drilling Sequence	41

# **Established 2001.**

Ufit Implant is a dental implant manufacturer and distributor with all the staff working hard in unison to provide the best product and the excellent service to dentists and patients all over the world.

With continued research and effort by the best technical researchers in Korea that started in 2001 to provide a world class product, the Sylbutment was developed and launched in 2010. With Ufit Implant continuous research, the solution for loosening of joined fixture and abutment was presented according to clinical studies. We are consistently researching for innovative technologies and high-quality products to provide the highest satisfaction.

Ufit Implant, as we have always been, promises to continually strive for excellence.

# The Ufit® Dental Implant History

## Beginings.

2001 JULY      Established T.STRONG

## Technology Development.

2004 FEB	Established T.STRONG INC.
2004 MAR	Acquired Patent Registration for Torque Wrench Driver Adapter
2004 MAY	Acquired Patent Registration for Dental Locking Abutment
2007 SEP	Acquired Certification from KOREA GOOD MANUFACTURING PRACTICE
2011 JUL	Received First Certified ISO 13485 License
2012 JAN	Received Domestic Patent for SYLBUTMENT

## Abroad.

2012 MAR	Attend PHARMED & HEALTHCARE VIETNAM
2013 JUL	Received Japan Patent for SYLBUTMENT
2013 SEP	Participate Moscow Dental Expo 2013 Moscow
2013 OCT	Participate World Dental Show(WDS) 2013 Mumbai
2014 JUN	Received U.S FDA Certificate
2014 DEC	Participate Greater New York Dental Meeting(GNYDM) 2014 New York
2015 JAN	Implamarket Became Kazakhstan Exclusive Distributor
2015 FEB	Received China Patent for Sylbutment
2015 MAR	Participate International Dental Show(IDS) 2015 German
2015 SEP	Participate FDI Annual Dental Congress 2015 Bangkok
MAR 2017	IDS Cologne - International dental show attended
MAR 2021	Ufit Implant is selected to pitch Deck for top 60 companies at MedTech Innovator
2021 Февраль	Implamarket Became Russia Exclusive Distributor

---

# Fixtures

**Nt2**



**Gt2**



# SYLBUTMENT

## A revolution in dental implant systems

SYLBUTMENT is the product of engineering research in which the perfect contact of two flat surfaces is only possible theoretically but practically impossible.



### Unprecedented - a remarkable sealing effect

The Sealing Effect occurs because of elastic modification done by the pressure on the circular bands of the contact sides.



### Outstanding durability due to even stress distribution

The even contact surfaces uniformly transfer power from prosthetic appliances to fixtures. Results of fatigue tests showed that not a single fatigue failure occurred when repeated high stress loads were applied.



### NO Gap

The circular bands act as a cushion within the limits of elastic modification when chewing force is applied.



### NO Loosening

The even surface contact of the circular grooved pattern evenly distributes chewing force within the limits of elasticity, preventing the screw from loosening and the abutment from swaying.


# The principles of SYLBUTMENT™

The principles of SYLBUTMENT are easily discovered around us.




## Conventional Abutment and SYLBUTMENT™

In a conventional abutment, the gap between fixture and abutment may increase gradually due to repeated chewing forces. This is due to the contact between the outer surface of the abutment and the inner taper of the fixture, which only occurs on a small surface area due to the roughness of both the two surfaces. On the other hand, SYLBUTMENT increases the contact surfaces by transformation to the grooves. It does not create a gap as the transformation between the two surfaces occurs within the elastic range.

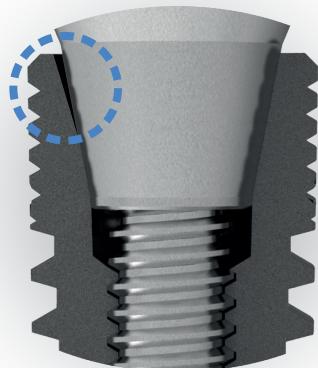


Conventional Abutment

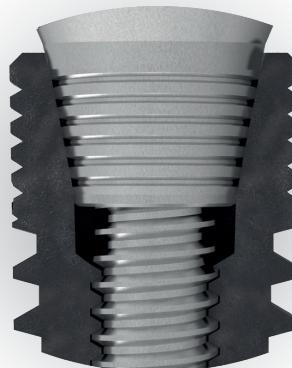

SYLBUTMENT™


## Why does loosening occur in conventional abutments?

The perfect contact of two flat surfaces is only possible theoretically but practically impossible.

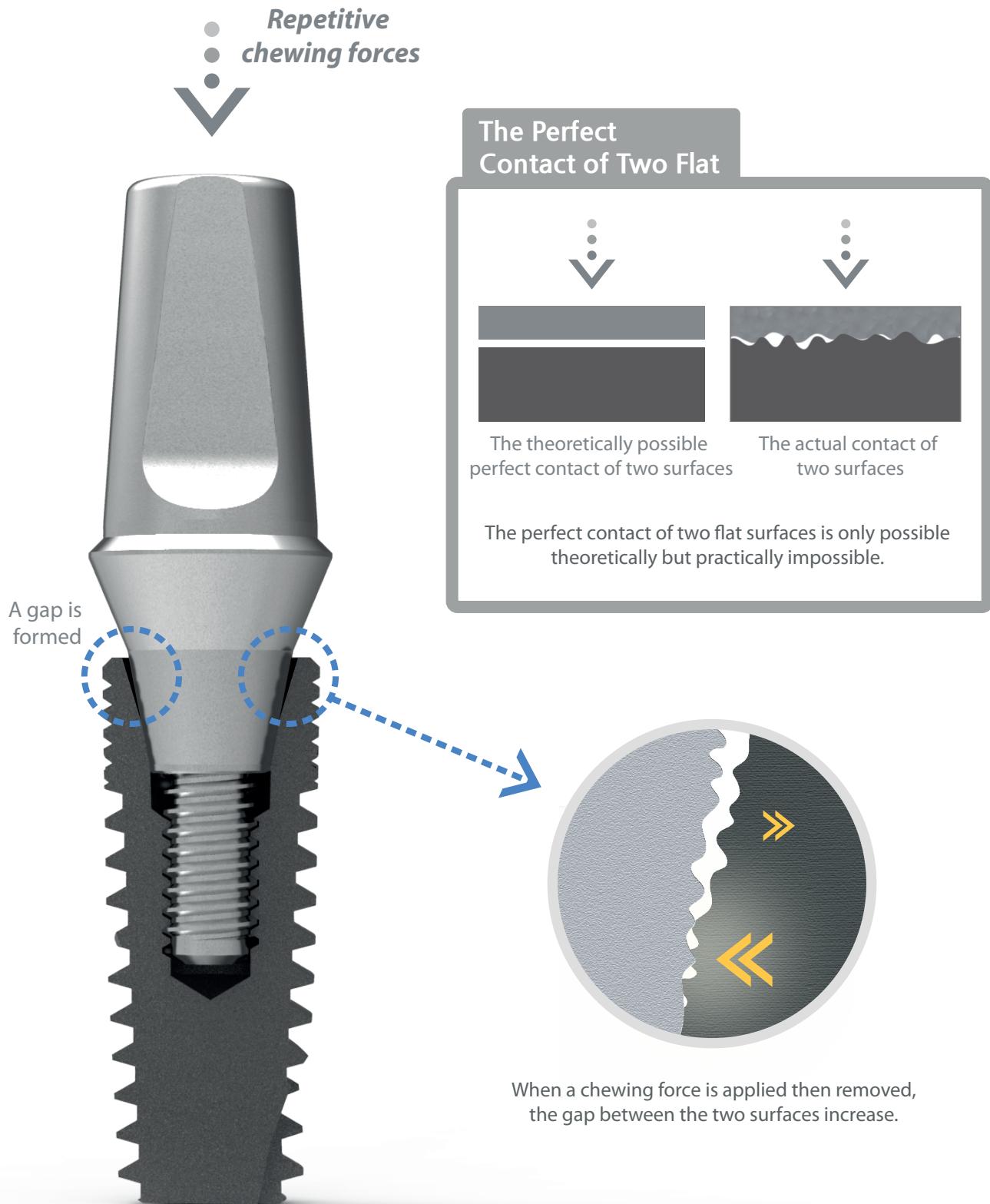


## The reason why **SYLBUTMENT™** is strong against fatigue (1)

When the abutment screw is fastened, elastic deformation occurs around the grooves of the SYLBUTMENT, creating a force which moves the abutment and fixture together.

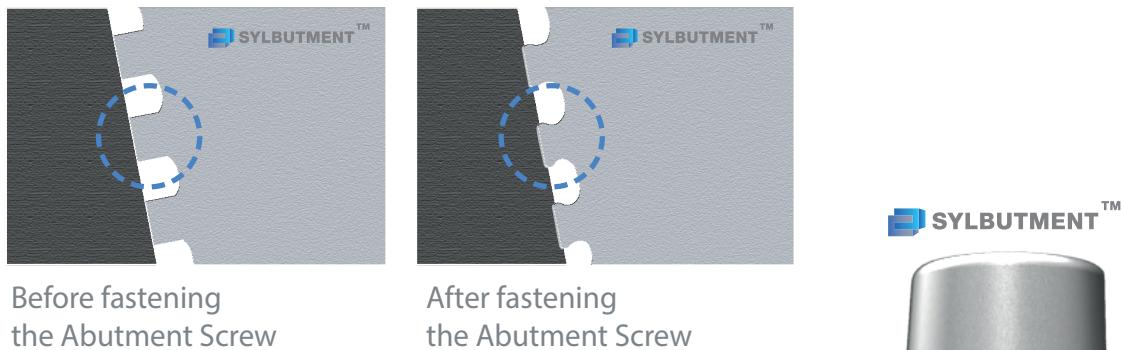


## The reason why **SYLBUTMENT™** is strong against fatigue (2)

As shown in the figure above, chewing forces are experienced asymmetrically due to the grooves of the SYLBUTMENT acting as an elastic body. This firmly maintains the sealed state of the abutment and distributes the chewing forces evenly in the fixture.

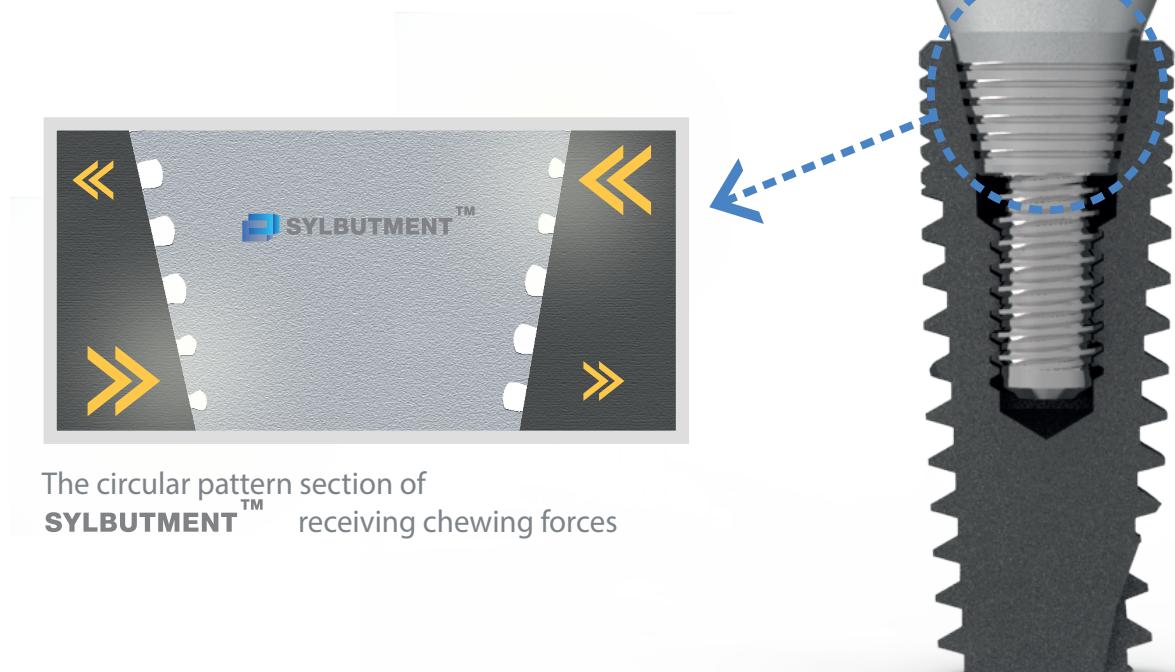


## Pressure distribution at the contact surfaces of the Fixtures and the Abutments (FEM Analysis)

When conventional abutments experience asymmetrical chewing forces, the contact surfaces of the fixtures and abutments are separated; however, when a SYLBUTMENT is used, the contact surfaces are not separated.

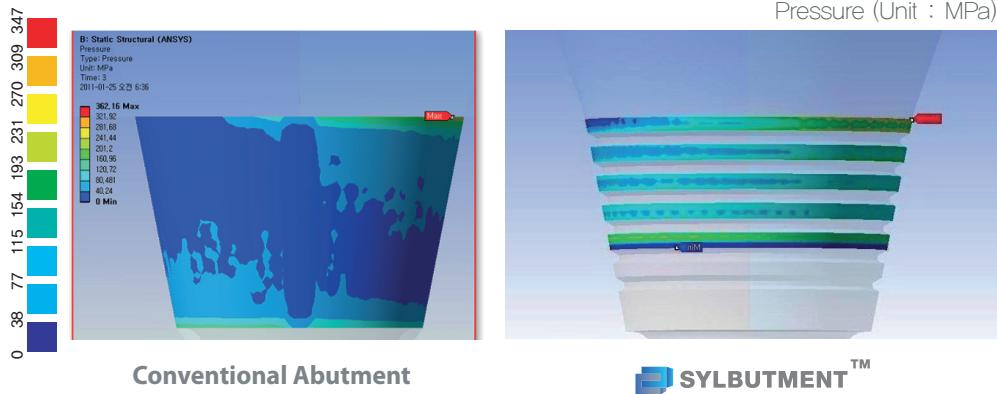

## Fatigue tests of the SYLBUTMENT™


Conventional Abutments can withstand 5 million repeated loads of 344N~34N, but the SYLBUTMENT can withstand 5 million repeated loads of 552N~55N.

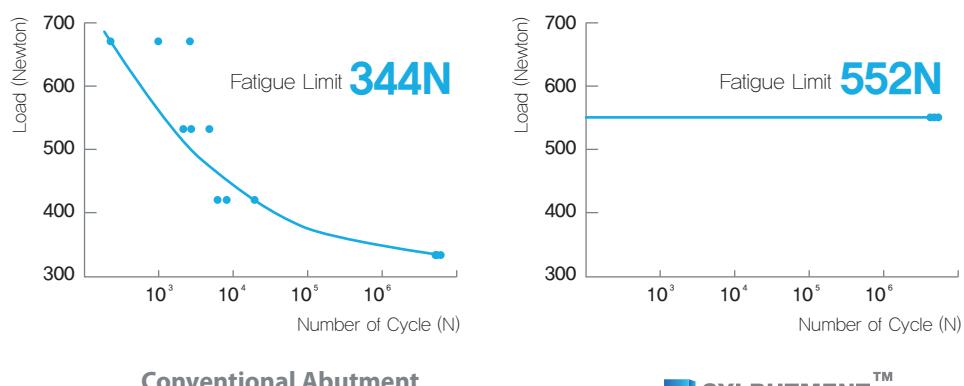

# Gt2 Fixtures

## Connection


2.5 Hex indentation and 11 degree Morse Taper.

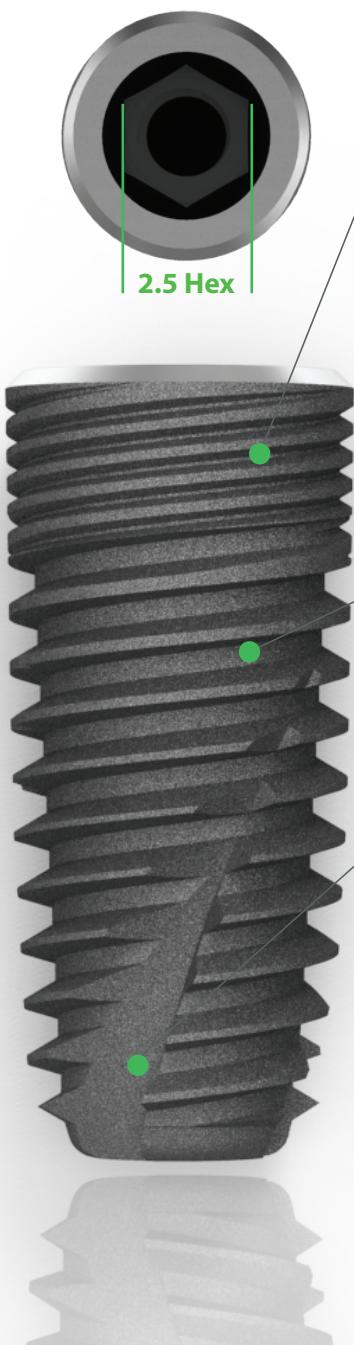

## Micro Thread


The deep 0.2 mm micro thread increases the surface area and induces a smooth connection with the larger main thread. Additionally, the micro thread increases thread contact with bone thereby improving the initial fixation effect.

## Dual Thread

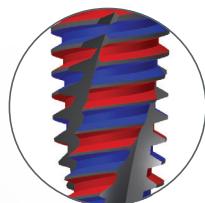

As 0.8mm pitch of dual thread type, the surgery time is reduced.  
(1.6mm per 1 rotation)

## RBM Surface

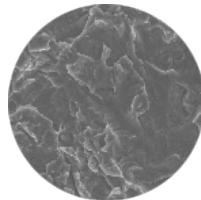


Surface areas are increased through blasting by highly biocompatible Calcium-Phosphate Media.

## Main Thread

When the fixture is inserted into the implant bed, the conical shape and lower deep thread of the fixture increase stability and make immediate loading possible.

## Cutting Edge

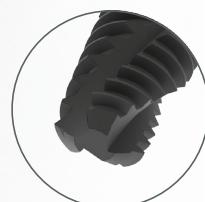

When placing the implants, the cutting edge of the Twist Type increases Self Tapping ability and minimizes Bone resistance.


## Apex

Apex has the dimension of  $D(\text{fixture diameter}) - 0.7\text{mm}$  and the body shape has the overall tapered one.


# Nt2 Fixtures

## Connection

2.5 Hex fastening Type of 11 degree Morse Taper Type

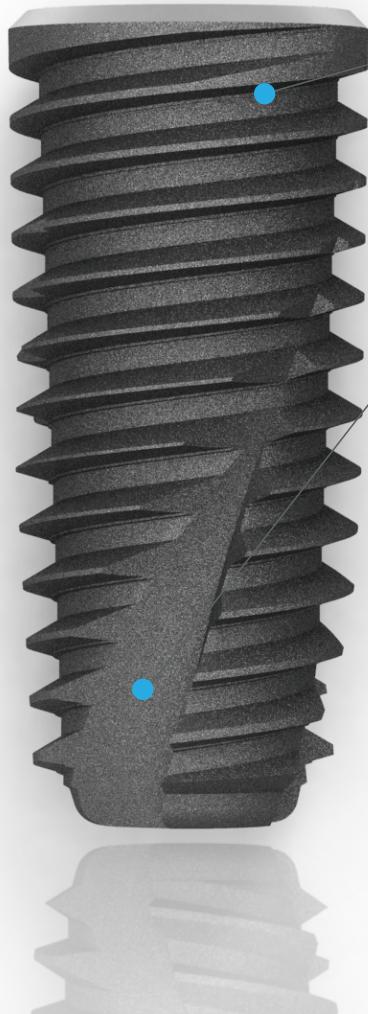


## Dual Thread

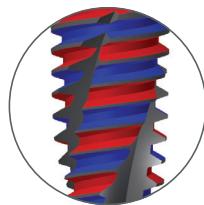

As 0.8 Dual Thread Type, the placing speed is very fast. (1.6mm per 1 rotation)

## RBM Surface

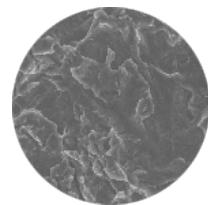


Surface areas are increased through blasting by highly biocompatible Calcium-Phosphate Media.

## Main Thread

When the fixture is inserted into the implant bed, the conical shape and lower deep thread of the fixture increase stability and make immediate loading possible.

## Cutting Edge

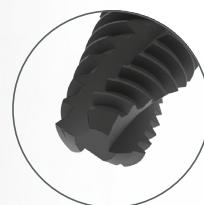


When placing the implants, the cutting edge of the Twist Type increases Self Tapping ability and minimizes Bone resistance.


## Apex

As a structure of D (Diameter) - 0.7mm, the overall Tapered type


# Submerged System Flow Chart

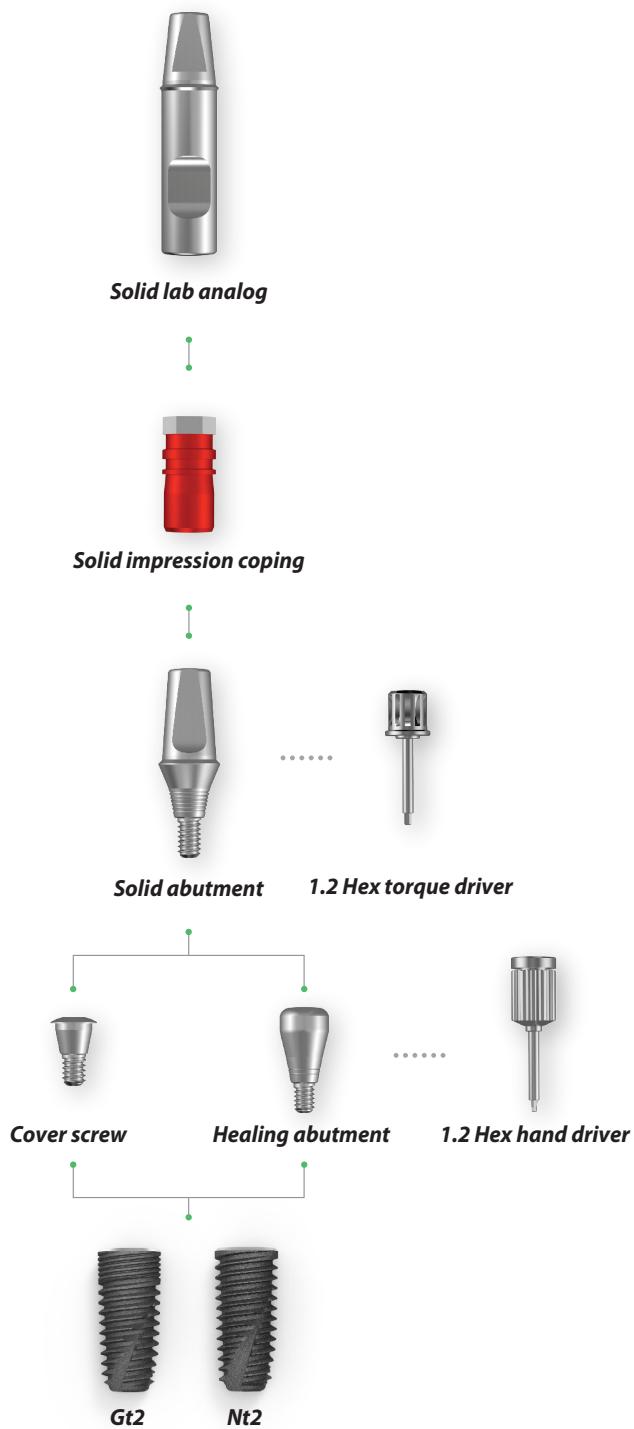

# Submerged System Flow Chart

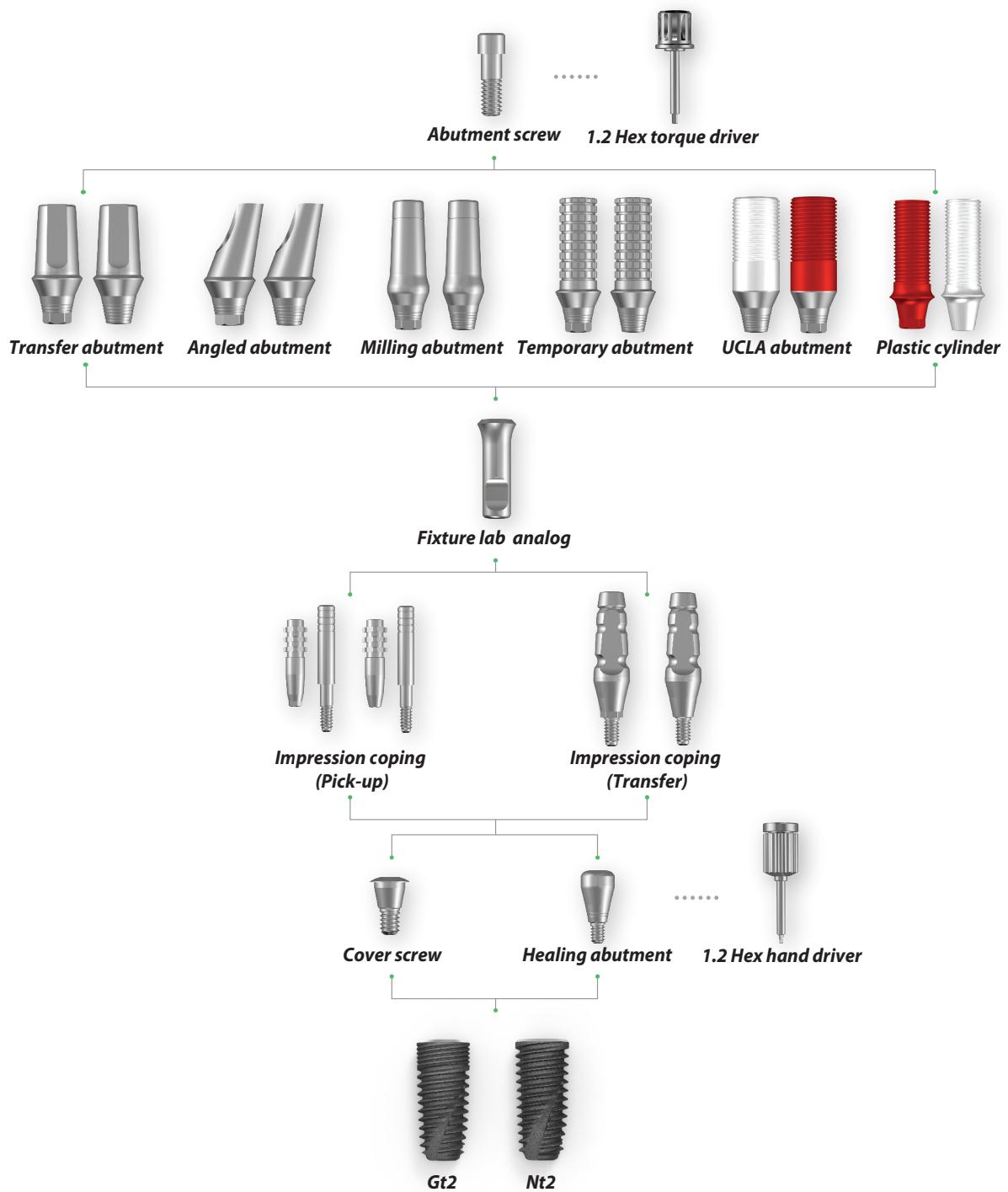

# Gt2 Fixture

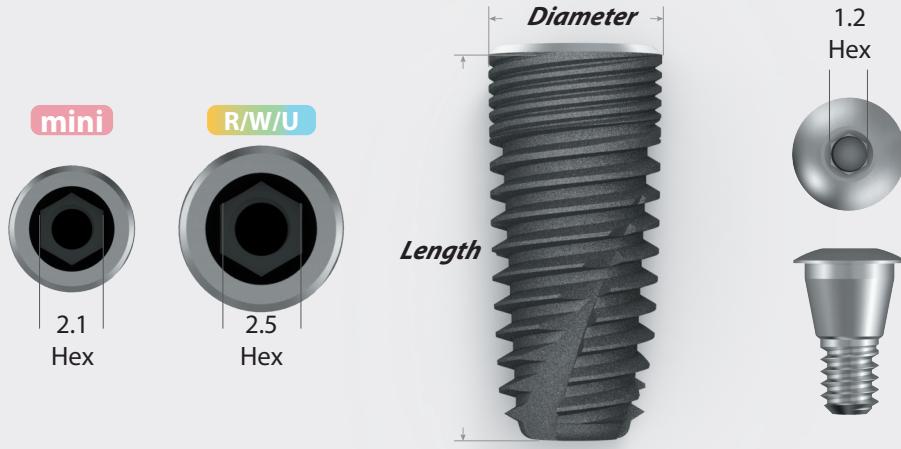

## Gt2 mini

**Length** ● **D3.5**

**8.5** GT2 35085 MT

**10** GT2 3510 MT

**11.5** GT2 35115 MT

**13** GT2 3513 MT

\* Use mini size abutment

- Mini
- Regular
- Wide
- Ultra-wide

## Gt2 R/W/U

**Length** ● **D4.0**

● **D4.5**

● **D5.0**

**7** GT2 4007 MT GT2 4507 MT GT2 5007 MT

**8.5** GT2 40085 MT GT2 45085 MT GT2 50085 MT

**10** GT2 4010 MT GT2 4510 MT GT2 5010 MT

**11.5** GT2 40115 MT GT2 45115 MT GT2 50115 MT

**13** GT2 4013 MT GT2 4513 MT GT2 5013 MT

**Length** ● **D5.5**

● **D6.0**

● **D6.5**

● **D7.0**

**7** GT2 5507 MT GT2 6007 MT GT2 6507 MT GT2 7007 MT

**8.5** GT2 55085 MT GT2 60085 MT GT2 65085 MT GT2 70085 MT


**10** GT2 5510 MT GT2 6010 MT GT2 6510 MT GT2 7010 MT

**11.5** GT2 55115 MT GT2 60115 MT GT2 65115 MT GT2 70115 MT

**13** GT2 5513 MT GT2 6013 MT GT2 6513 MT GT2 7013 MT

\* Use R/W/U size abutment

# Nt2 Fixture

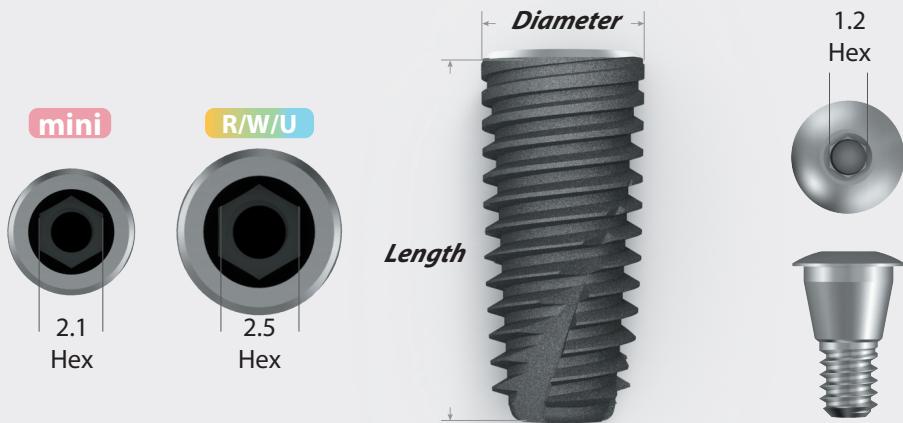

## Nt2 mini

**Length** ● **D3.5**

<b>8.5</b>	NT2 35085 T
<b>10</b>	NT2 3510 T
<b>11.5</b>	NT2 35115 T
<b>13</b>	NT2 3513 T

\* Use mini size abutment

- Mini
- Regular
- Wide
- Ultra-wide

## Nt2 R/W/U

<b>Length</b>	● <b>D4.0</b>	● <b>D4.5</b>	● <b>D5.0</b>
<b>7</b>	NT2 4007 T	NT2 4507 T	NT2 5007 T
<b>8.5</b>	NT2 40085 T	NT2 45085 T	NT2 50085 T
<b>10</b>	NT2 4010 T	NT2 4510 T	NT2 5010 T
<b>11.5</b>	NT2 40115 T	NT2 45115 T	NT2 50115 T
<b>13</b>	NT2 4013 T	NT2 4513 T	NT2 5013 T

<b>Length</b>	● <b>D5.5</b>	● <b>D6.0</b>	● <b>D6.5</b>	● <b>D7.0</b>
<b>7</b>	NT2 5507 T	NT2 6007 T	NT2 6507 T	NT2 7007 T
<b>8.5</b>	NT2 55085 T	NT2 60085 T	NT2 65085 T	NT2 70085 T
<b>10</b>	NT2 5510 T	NT2 6010 T	NT2 6510 T	NT2 7010 T
<b>11.5</b>	NT2 55115 T	NT2 60115 T	NT2 65115 T	NT2 70115 T
<b>13</b>	NT2 5513 T	NT2 6013 T	NT2 6513 T	NT2 7013 T

\* Use R/W/U size abutment

# Submerged Abutment

- Mini
- Regular
- Wide
- Ultra-wide

*Height* 

## Cover screw mini

### Height

0.5	MICS 5005
2	MICS 5020

## Cover screw R/W/U

### Height


0.5	22HCSR 5005
2	22HCSR 5020

### Method

Use 1.2 Hex hand driver  
5~8Ncm Joining torque

### Usage

Used to prevent foreign materials from entering after the fixture insertion

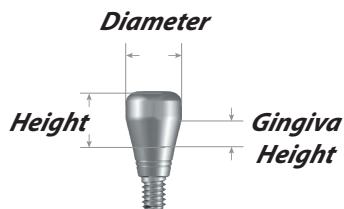

### Healing abutment mini

	<i>GH</i>	<i>Height 3</i>	<i>Height 4</i>	<i>Height 5</i>	<i>Height 7</i>
<b>D4.0</b>	<b>1</b>	MHA 4013			
	<b>2</b>		MHA 4024	MHA 4025	
	<b>3</b>				MHA 4037
<b>D4.5</b>	<b>1</b>	MHA 4513			
	<b>2</b>		MHA 4524	MHA 4525	
	<b>3</b>				MHA 4537

### Healing abutment R/W/U


	<i>GH</i>	<i>Height 3</i>	<i>Height 4</i>	<i>Height 5</i>	<i>Height 7</i>
<b>D4.0</b>	<b>1</b>	SHA 401030			
	<b>2</b>		SHA 402040	SHA 402050	
	<b>3</b>				SHA 403070
<b>D4.5</b>	<b>1</b>	SHA 451030			
	<b>2</b>		SHA 452040	SHA 452050	
	<b>3</b>				SHA 453070
<b>D5.0</b>	<b>1</b>	SHA 501030			
	<b>2</b>		SHA 502040	SHA 502050	
	<b>3</b>				SHA 503070
<b>D5.5</b>	<b>1</b>	SHA 551030			
	<b>2</b>		SHA 552040	SHA 552050	
	<b>3</b>				SHA 553070
<b>D6.0</b>	<b>1</b>	SHA 601030			
	<b>2</b>		SHA 602040	SHA 602050	
	<b>3</b>				SHA 603070
<b>D6.5</b>	<b>1</b>	SHA 651030			
	<b>2</b>		SHA 652040	SHA 652050	
	<b>3</b>				SHA 653070

#### Method

Use 1.2 Hex hand driver  
5~8Ncm of joining torque

#### Usage

Used to protect the connecting part of the implant  
Acts as the shape of the gingiva after surgery  
Abutment is chosen according to the patient's gingival height.

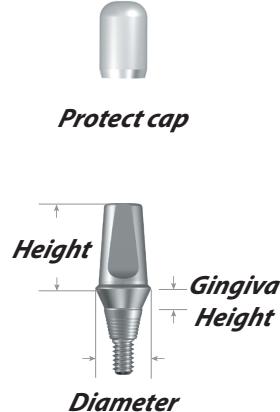

### Solid abutment mini

 SYLBUTMENT™

	<b>H</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>4</b>	MSA 4014	MSA 4024	MSA 4034	MSA 4044	MSA 4054
	<b>5.5</b>	MSA 4015	MSA 4025	MSA 4035	MSA 4045	MSA 4055
	<b>7</b>	MSA 4017	MSA 4027	MSA 4037	MSA 4047	MSA 4057
<b>D4.5</b>	<b>4</b>	MSA 4514	MSA 4524	MSA 4534	MSA 4544	MSA 4554
	<b>5.5</b>	MSA 4515	MSA 4525	MSA 4535	MSA 4545	MSA 4555
	<b>7</b>	MSA 4517	MSA 4527	MSA 4537	MSA 4547	MSA 4557

### Solid abutment R/W/U

 SYLBUTMENT™

	<b>H</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>4</b>	SSA 401040	SSA 402040	SSA 403040	SSA 404040	SSA 405040
	<b>5.5</b>	SSA 401055	SSA 402055	SSA 403055	SSA 404055	SSA 405055
	<b>7</b>	SSA 401070	SSA 402070	SSA 403070	SSA 404070	SSA 405070
<b>D4.5</b>	<b>4</b>	SSA 451040	SSA 452040	SSA 453040	SSA 454040	SSA 455040
	<b>5.5</b>	SSA 451055	SSA 452055	SSA 453055	SSA 454055	SSA 455055
	<b>7</b>	SSA 451070	SSA 452070	SSA 453070	SSA 454070	SSA 455070
<b>D5.0</b>	<b>4</b>	SSA 501040	SSA 502040	SSA 503040	SSA 504040	SSA 505040
	<b>5.5</b>	SSA 501055	SSA 502055	SSA 503055	SSA 504055	SSA 505055
	<b>7</b>	SSA 501070	SSA 502070	SSA 503070	SSA 504070	SSA 505070
<b>D5.5</b>	<b>4</b>	SSA 551040	SSA 552040	SSA 553040	SSA 554040	SSA 555040
	<b>5.5</b>	SSA 551055	SSA 552055	SSA 553055	SSA 554055	SSA 555055
	<b>7</b>	SSA 551070	SSA 552070	SSA 553070	SSA 554070	SSA 555070
<b>D6.0</b>	<b>4</b>	SSA 601040	SSA 602040	SSA 603040	SSA 604040	SSA 605040
	<b>5.5</b>	SSA 601055	SSA 602055	SSA 603055	SSA 604055	SSA 605055
	<b>7</b>	SSA 601070	SSA 602070	SSA 603070	SSA 604070	SSA 605070
<b>D6.5</b>	<b>4</b>	SSA 651040	SSA 652040	SSA 653040	SSA 654040	SSA 655040
	<b>5.5</b>	SSA 651055	SSA 652055	SSA 653055	SSA 654055	SSA 655055
	<b>7</b>	SSA 651070	SSA 652070	SSA 653070	SSA 654070	SSA 655070

#### Method



Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Solid abutment + Protect cap

#### Usage

Used on the conventional cement type produced prosthesis  
All-in-one abutment and screw structure

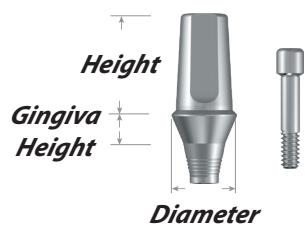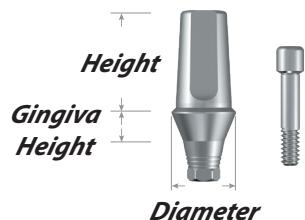

### Transfer abutment Hex mini



	<i>H</i>	<i>GH 1</i>	<i>GH 2</i>	<i>GH 3</i>	<i>GH 4</i>	<i>GH 5</i>
<b>D4.0</b>	<b>4</b>	MTA 4014H	MTA 4024H	MTA 4034H	MTA 4044H	MTA 4054H
	<b>5.5</b>	MTA 4015H	MTA 4025H	MTA 4035H	MTA 4045H	MTA 4055H
	<b>7</b>	MTA 4017H	MTA 4027H	MTA 4037H	MTA 4047H	MTA 4057H
<b>D4.5</b>	<b>4</b>	MTA 4514H	MTA 4524H	MTA 4534H	MTA 4544H	MTA 4554H
	<b>5.5</b>	MTA 4515H	MTA 4525H	MTA 4535H	MTA 4545H	MTA 4555H
	<b>7</b>	MTA 4517H	MTA 4527H	MTA 4537H	MTA 4547H	MTA 4557H

### Transfer abutment Non-Hex mini



	<i>H</i>	<i>GH 1</i>	<i>GH 2</i>	<i>GH 3</i>	<i>GH 4</i>	<i>GH 5</i>
<b>D4.0</b>	<b>4</b>	MTA 4014N	MTA 4024N	MTA 4034N	MTA 4044N	MTA 4054N
	<b>5.5</b>	MTA 4015N	MTA 4025N	MTA 4035N	MTA 4045N	MTA 4055N
	<b>7</b>	MTA 4017N	MTA 4027N	MTA 4037N	MTA 4047N	MTA 4057N
<b>D4.5</b>	<b>4</b>	MTA 4514N	MTA 4524N	MTA 4534N	MTA 4544N	MTA 4554N
	<b>5.5</b>	MTA 4515N	MTA 4525N	MTA 4535N	MTA 4545N	MTA 4555N
	<b>7</b>	MTA 4517N	MTA 4527N	MTA 4537N	MTA 4547N	MTA 4557N

#### Method

Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Transfer abutment + Abutment screw  
Choice of variety of sizes according to gingival height

#### Usage

Conventional cement retained type abutment



### Transfer abutment Hex R/W/U

SYLBUTMENT™

	<i>H</i>	<i>GH 1</i>	<i>GH 2</i>	<i>GH 3</i>	<i>GH 4</i>	<i>GH 5</i>
	<b>4</b>	STA 401040H	STA 402040H	STA 403040H	STA 404040H	STA 405040H
<b>D4.0</b>	<b>5.5</b>	STA 401055H	STA 402055H	STA 403055H	STA 404055H	STA 405055H
	<b>7</b>	STA 401070H	STA 402070H	STA 403070H	STA 404070H	STA 405070H
	<b>4</b>	STA 451040H	STA 452040H	STA 453040H	STA 454040H	STA 455040H
<b>D4.5</b>	<b>5.5</b>	STA 451055H	STA 452055H	STA 453055H	STA 454055H	STA 455055H
	<b>7</b>	STA 451070H	STA 452070H	STA 453070H	STA 454070H	STA 455070H
	<b>4</b>	STA 501040H	STA 502040H	STA 503040H	STA 504040H	STA 505040H
<b>D5.0</b>	<b>5.5</b>	STA 501055H	STA 502055H	STA 503055H	STA 504055H	STA 505055H
	<b>7</b>	STA 501070H	STA 502070H	STA 503070H	STA 504070H	STA 505070H
	<b>4</b>	STA 551040H	STA 552040H	STA 553040H	STA 554040H	STA 555040H
<b>D5.5</b>	<b>5.5</b>	STA 551055H	STA 552055H	STA 553055H	STA 554055H	STA 555055H
	<b>7</b>	STA 551070H	STA 552070H	STA 553070H	STA 554070H	STA 555070H
	<b>4</b>	STA 601040H	STA 602040H	STA 603040H	STA 604040H	STA 605040H
<b>D6.0</b>	<b>5.5</b>	STA 601055H	STA 602055H	STA 603055H	STA 604055H	STA 605055H
	<b>7</b>	STA 601070H	STA 602070H	STA 603070H	STA 604070H	STA 605070H
	<b>4</b>	STA 651040H	STA 652040H	STA 653040H	STA 654040H	STA 655040H
<b>D6.5</b>	<b>5.5</b>	STA 651055H	STA 652055H	STA 653055H	STA 654055H	STA 655055H
	<b>7</b>	STA 651070H	STA 652070H	STA 653070H	STA 654070H	STA 655070H

#### Method


Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Transfer abutment + Abutment screw  
Choice of variety of sizes according to gingival height

#### Usage

Conventional cement retained type abutment

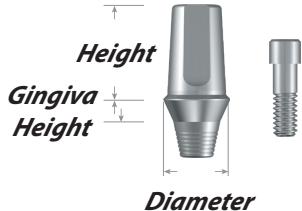

### Transfer abutment Non-Hex R/W/U

SYLBUTMENT™

	<i>H</i>	<i>GH 1</i>	<i>GH 2</i>	<i>GH 3</i>	<i>GH 4</i>	<i>GH 5</i>
<b>D4.0</b>	<b>4</b>	STA 401040N	STA 402040N	STA 403040N	STA 404040N	STA 405040N
	<b>5.5</b>	STA 401055N	STA 402055N	STA 403055N	STA 404055N	STA 405055N
	<b>7</b>	STA 401070N	STA 402070N	STA 403070N	STA 404070N	STA 405070N
<b>D4.5</b>	<b>4</b>	STA 451040N	STA 452040N	STA 453040N	STA 454040N	STA 455040N
	<b>5.5</b>	STA 451055N	STA 452055N	STA 453055N	STA 454055N	STA 455055N
	<b>7</b>	STA 451070N	STA 452070N	STA 453070N	STA 454070N	STA 455070N
<b>D5.0</b>	<b>4</b>	STA 501040N	STA 502040N	STA 503040N	STA 504040N	STA 505040N
	<b>5.5</b>	STA 501055N	STA 502055N	STA 503055N	STA 504055N	STA 505055N
	<b>7</b>	STA 501070N	STA 502070N	STA 503070N	STA 504070N	STA 505070N
<b>D5.5</b>	<b>4</b>	STA 551040N	STA 552040N	STA 553040N	STA 554040N	STA 555040N
	<b>5.5</b>	STA 551055N	STA 552055N	STA 553055N	STA 554055N	STA 555055N
	<b>7</b>	STA 551070N	STA 552070N	STA 553070N	STA 554070N	STA 555070N
<b>D6.0</b>	<b>4</b>	STA 601040N	STA 602040N	STA 603040N	STA 604040N	STA 605040N
	<b>5.5</b>	STA 601055N	STA 602055N	STA 603055N	STA 604055N	STA 605055N
	<b>7</b>	STA 601070N	STA 602070N	STA 603070N	STA 604070N	STA 605070N
<b>D6.5</b>	<b>4</b>	STA 651040N	STA 652040N	STA 653040N	STA 654040N	STA 655040N
	<b>5.5</b>	STA 651055N	STA 652055N	STA 653055N	STA 654055N	STA 655055N
	<b>7</b>	STA 651070N	STA 652070N	STA 653070N	STA 654070N	STA 655070N

#### Method



Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Transfer abutment + Abutment screw  
Choice of variety of sizes according to gingival height

#### Usage

Conventional cement retained type abutment

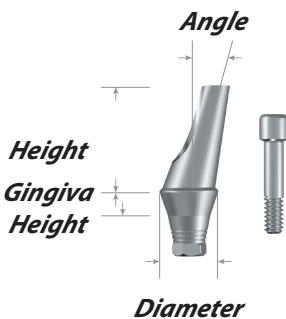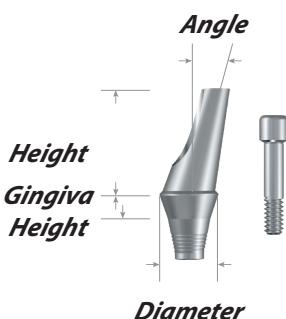Angled abutment Hex mini					
<b>A Type</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>15</b> MAA 4011A	MAA 4021A	MAA 4031A	MAA 4041A	MAA 4051A
	<b>25</b> MAA 4012A	MAA 4022A	MAA 4032A	MAA 4042A	MAA 4052A
<b>D4.5</b>	<b>15</b> MAA 4511A	MAA 4521A	MAA 4531A	MAA 4541A	MAA 4551A
	<b>25</b> MAA 4512A	MAA 4522A	MAA 4532A	MAA 4542A	MAA 4552A

<b>B type</b>	<b>G/H 1</b>	<b>G/H 2</b>	<b>G/H 3</b>	<b>G/H 4</b>	<b>G/H 5</b>
<b>D4.0</b>	<b>15</b> MAA 4011B	MAA 4021B	MAA 4031B	MAA 4041B	MAA 4051B
	<b>25</b> MAA 4012B	MAA 4022B	MAA 4032B	MAA 4042B	MAA 4052B
<b>D4.5</b>	<b>15</b> MAA 4511B	MAA 4521B	MAA 4531B	MAA 4541B	MAA 4551B
	<b>25</b> MAA 4512B	MAA 4522B	MAA 4532B	MAA 4542B	MAA 4552B

H = 7mm

Angled abutment Non-Hex mini					
<b>A</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>15</b> MAA 4011N	MAA 4021N	MAA 4031N	MAA 4041N	MAA 4051N
	<b>25</b> MAA 4012N	MAA 4022N	MAA 4032N	MAA 4042N	MAA 4052N
<b>D4.5</b>	<b>15</b> MAA 4511N	MAA 4521N	MAA 4531N	MAA 4541N	MAA 4551N
	<b>25</b> MAA 4512N	MAA 4522N	MAA 4532N	MAA 4542N	MAA 4552N

H = 7mm

#### Method


Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Angled abutment + Abutment screw  
15° / 25° composition

#### Usage

Conventional cement retained type abutment  
Used in revising the fixture's path  
Used in cases when the prosthesis' path needs to be adjusted

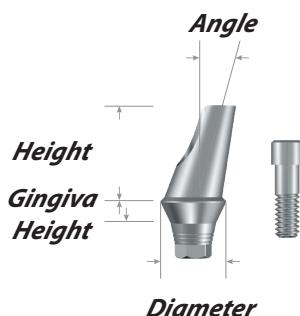

### Angled abutment Hex R/W/U

SYLBUTMENT™

<b>A Type</b>	<b>A</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>15</b>	SAA 401015A	SAA 402015A	SAA 403015A	SAA 404015A	SAA 405015A
	<b>25</b>	SAA 401025A	SAA 402025A	SAA 403025A	SAA 404025A	SAA 405025A
<b>D4.5</b>	<b>15</b>	SAA 451015A	SAA 452015A	SAA 453015A	SAA 454015A	SAA 455015A
	<b>25</b>	SAA 451025A	SAA 452025A	SAA 453025A	SAA 454025A	SAA 455025A
<b>D5.0</b>	<b>15</b>	SAA 501015A	SAA 502015A	SAA 503015A	SAA 504015A	SAA 505015A
	<b>25</b>	SAA 501025A	SAA 502025A	SAA 503025A	SAA 504025A	SAA 505025A
<b>D5.5</b>	<b>15</b>	SAA 551015A	SAA 552015A	SAA 553015A	SAA 554015A	SAA 555015A
	<b>25</b>	SAA 551025A	SAA 552025A	SAA 553025A	SAA 554025A	SAA 555025A
<b>D6.0</b>	<b>15</b>	SAA 601015A	SAA 602015A	SAA 603015A	SAA 604015A	SAA 605015A
	<b>25</b>	SAA 601025A	SAA 602025A	SAA 603025A	SAA 604025A	SAA 605025A
<b>D6.5</b>	<b>15</b>	SAA 651015A	SAA 652015A	SAA 653015A	SAA 654015A	SAA 655015A
	<b>25</b>	SAA 651025A	SAA 652025A	SAA 653025A	SAA 654025A	SAA 655025A

<b>B type</b>	<b>A</b>	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.0</b>	<b>15</b>	SAA 401015B	SAA 402015B	SAA 403015B	SAA 404015B	SAA 405015B
	<b>25</b>	SAA 401025B	SAA 402025B	SAA 403025B	SAA 404025B	SAA 405025B
<b>D4.5</b>	<b>15</b>	SAA 451015B	SAA 452015B	SAA 453015B	SAA 454015B	SAA 455015B
	<b>25</b>	SAA 451025B	SAA 452025B	SAA 453025B	SAA 454025B	SAA 455025B
<b>D5.0</b>	<b>15</b>	SAA 501015B	SAA 502015B	SAA 503015B	SAA 504015B	SAA 505015B
	<b>25</b>	SAA 501025B	SAA 502025B	SAA 503025B	SAA 504025B	SAA 505025B
<b>D5.5</b>	<b>15</b>	SAA 551015B	SAA 552015B	SAA 553015B	SAA 554015B	SAA 555015B
	<b>25</b>	SAA 551025B	SAA 552025B	SAA 553025B	SAA 554025B	SAA 555025B
<b>D6.0</b>	<b>15</b>	SAA 601015B	SAA 602015B	SAA 603015B	SAA 604015B	SAA 605015B
	<b>25</b>	SAA 601025B	SAA 602025B	SAA 603025B	SAA 604025B	SAA 605025B
<b>D6.5</b>	<b>15</b>	SAA 651015B	SAA 652015B	SAA 653015B	SAA 654015B	SAA 655015B
	<b>25</b>	SAA 651025B	SAA 652025B	SAA 653025B	SAA 654025B	SAA 655025B

H = 7mm

#### Method


Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Angled abutment + Abutment screw  
15° / 25° composition

#### Usage

Conventional cement retained type abutment  
Used in revising the fixture's path  
Used in cases when the prosthesis' path needs to be adjusted

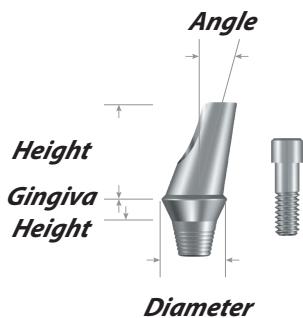

### Angled abutment Non-Hex R/W/U

SYLBUTMENT™

	A	GH 1	GH 2	GH 3	GH 4	GH 5
<b>D4.0</b>	<b>15</b>	SAA 401015N	SAA 402015N	SAA 403015N	SAA 404015N	SAA 405015N
	<b>25</b>	SAA 401025N	SAA 402025N	SAA 403025N	SAA 404025N	SAA 405025N
<b>D4.5</b>	<b>15</b>	SAA 451015N	SAA 452015N	SAA 453015N	SAA 454015N	SAA 455015N
	<b>25</b>	SAA 451025N	SAA 452025N	SAA 453025N	SAA 454025N	SAA 455025N
<b>D5.0</b>	<b>15</b>	SAA 501015N	SAA 502015N	SAA 503015N	SAA 504015N	SAA 505015N
	<b>25</b>	SAA 501025N	SAA 502025N	SAA 503025N	SAA 504025N	SAA 505025N
<b>D5.5</b>	<b>15</b>	SAA 551015N	SAA 552015N	SAA 553015N	SAA 554015N	SAA 555015N
	<b>25</b>	SAA 551025N	SAA 552025N	SAA 553025N	SAA 554025N	SAA 555025N
<b>D6.0</b>	<b>15</b>	SAA 601015N	SAA 602015N	SAA 603015N	SAA 604015N	SAA 605015N
	<b>25</b>	SAA 601025N	SAA 602025N	SAA 603025N	SAA 604025N	SAA 605025N
<b>D6.5</b>	<b>15</b>	SAA 651015N	SAA 652015N	SAA 653015N	SAA 654015N	SAA 655015N
	<b>25</b>	SAA 651025N	SAA 652025N	SAA 653025N	SAA 654025N	SAA 655025N

H = 7mm

#### Method





Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Angled abutment + Abutment screw  
15° / 25° composition

#### Usage

Conventional cement retained type abutment  
Used in revising the fixture's path  
Used in cases when the prosthesis' path needs to be adjusted

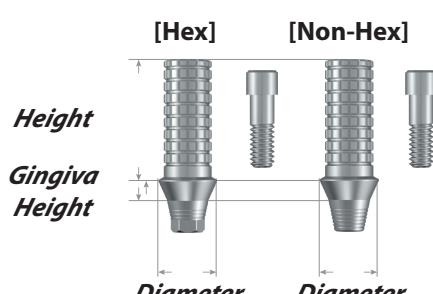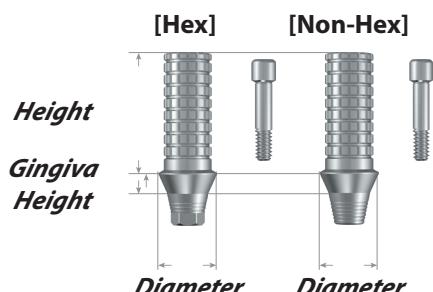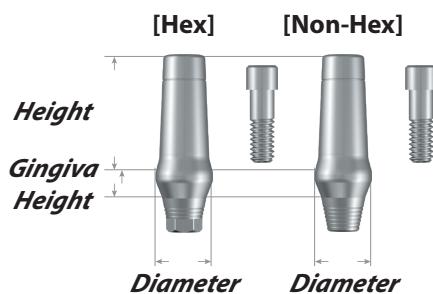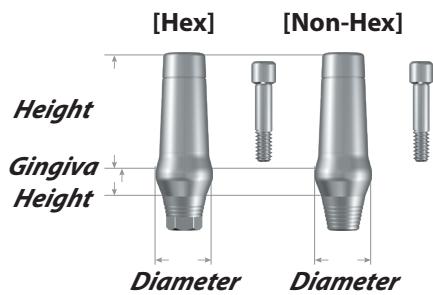

### Milling abutment **mini**

SYLBUTMENT™

#### **GH 2**

#### **GH 4**

	<b>Hex</b>	<b>Non-Hex</b>	<b>Hex</b>	<b>Non-Hex</b>
<b>D4.5</b>	MMA 4529H	MMA 4529N	MMA 4549H	MMA 4549N

### Milling abutment **R/W/U**

SYLBUTMENT™

#### **GH 2**

#### **GH 4**

	<b>Hex</b>	<b>Non-Hex</b>	<b>Hex</b>	<b>Non-Hex</b>
<b>D5.0</b>	SMA 5029H	SMA 5029N	SMA 5049H	SMA 5049N
<b>D5.5</b>	SMA 5529H	SMA 5529N	SMA 5549H	SMA 5549N
<b>D6.0</b>	SMA 6029H	SMA 6029N	SMA 6049H	SMA 6049N

**H = 9mm**

#### Method

Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Milling abutment + Abutment screw

#### Usage

Used in cases when the height or margin of abutment needs to be customized

### Temporary abutment **mini**

SYLBUTMENT™

#### **GH 2**

#### **GH 4**

	<b>Hex</b>	<b>Non-Hex</b>	<b>Hex</b>	<b>Non-Hex</b>
<b>D4.5</b>	MTPA 452H	MTPA 452N	MTPA 454H	MTPA 454N

### Temporary abutment **R/W/U**

SYLBUTMENT™

#### **GH 2**

#### **GH 4**

	<b>Hex</b>	<b>Non-Hex</b>	<b>Hex</b>	<b>Non-Hex</b>
<b>D5.0</b>	STPA 502H	STPA 502N	STPA 504H	STPA 504N

**H = 10mm**

#### Method


Use 1.2 Hex torque driver  
25~35Ncm joining torque

#### Components

Temporary abutment + Abutment screw

#### Usage

Used in cases making the temporary prosthesis

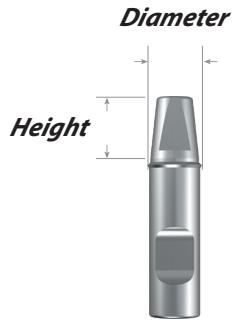

### Solid lab analog M/R/W/U

	Height 4	Height 5.5	Height 7
D4.0	S-SLA 4040	S-SLA 4055	S-SLA 4070
D4.5	S-SLA 4540	S-SLA 4555	S-SLA 4570
D5.0	S-SLA 5040	S-SLA 5055	S-SLA 5070
D5.5	S-SLA 5540	S-SLA 5555	S-SLA 5570
D6.0	S-SLA 6040	S-SLA 6055	S-SLA 6070
D6.5	S-SLA 6540	S-SLA 6555	S-SLA 6570


#### Method

Used on solid abutment features

Used to produce the model for solid Impression coping connection pick up inside the oral cavity

#### Usage

Solid abutment is materialized in the oral cavity on the working replica

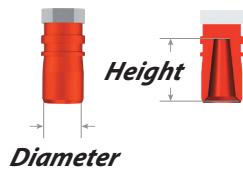

### Solid impression coping M/R/W/U

	Height 4	Height 5.5	Height 7
D4.0	S-IC 4040	S-IC 4055	S-IC 4070
D4.5	S-IC 4540	S-IC 4555	S-IC 4570
D5.0	S-IC 5040	S-IC 5055	S-IC 5070
D5.5	S-IC 5540	S-IC 5555	S-IC 5570
D6.0	S-IC 6040	S-IC 6055	S-IC 6070
D6.5	S-IC 6540	S-IC 6555	S-IC 6570

#### Method

Used on solid Abutment features

Integration of existing positioning cylinder and Impression Cap



### Fixture lab analog mini

M-FLA 35

### Fixture lab analog R/W/U

S-FLA 45

#### Method

For Gt2/Nt2



Used on abutment features

Used to produce the model for solid Impression coping

Connection pick up inside the oral cavity

#### Usage

Fixture is materialized in the oral cavity on the working replica

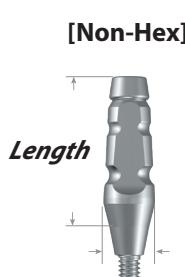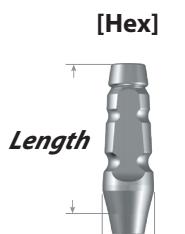

### Impression coping (Transfer) mini

<i>Length 11</i>		<i>Length 15</i>	
<i>Hex</i>	<i>Non-Hex</i>	<i>Hex</i>	<i>Non-Hex</i>
<b>D4.0</b>	M-ICT 4011H M-ICT 4011N M-ICT 4015H M-ICT 4015N		

### Impression coping (Transfer) R/W/U

<i>Length 11</i>		<i>Length 15</i>	
<i>Hex</i>	<i>Non-Hex</i>	<i>Hex</i>	<i>Non-Hex</i>
<b>D4.5</b>	S-ICT 4511H S-ICT 4511N S-ICT 4515H S-ICT 4515N		

#### Method

Use 1.2 Hex hand driver

#### Components


Impression coping + Guide pin (2 pieces)

11mm / 15mm Coping size


#### Usage

Existing tray is used

[Hex]

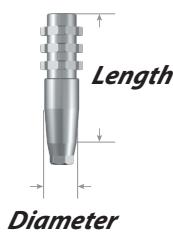

[Non-Hex]

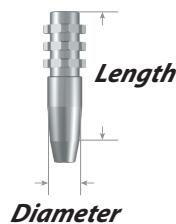

### Impression coping (Pick-up) mini

Length 10		Length 15		
Hex	Non-Hex	Hex	Non-Hex	
<b>D4.0</b>	M-ICP 4010H	M-ICP 4010N	M-ICP 4015H	M-ICP 4015N

### Impression coping (Pick-up) R/W/U

Length 10		Length 15		
Hex	Non-Hex	Hex	Non-Hex	
<b>D4.0</b>	S-ICP 4010H	S-ICP 4010N	S-ICP 4015H	S-ICP 4015N
<b>D4.5</b>	S-ICP 4510H	S-ICP 4510N	S-ICP 4515H	S-ICP 4515N
<b>D5.0</b>	S-ICP 5010H	S-ICP 5010N	S-ICP 5015H	S-ICP 5015N
<b>D5.5</b>	S-ICP 5510H	S-ICP 5510N	S-ICP 5515H	S-ICP 5515N
<b>D6.0</b>	S-ICP 6010H	S-ICP 6010N	S-ICP 6015H	S-ICP 6015N

### Impression coping Guide pin (Pick-up) mini

Length 10	Length 15	Length 20
M-PG 100	M-PG 150	M-PG 200

### Impression coping Guide pin (Pick-up) R/W/U

Length 10	Length 15	Length 20
S-PG 100	S-PG 150	S-PG 200

#### Method


Use hand or 1.2 Hex hand driver

#### Components

Impression coping + Guide pin  
10mm/15mm/20mm Guide pin size

#### Usage


Use of custom tray  
Increases the ease of various guide pin size

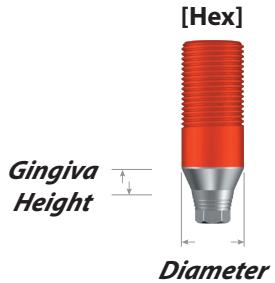**UCLA Abutment mini**

<i>Hex</i>	<i>Non-Hex</i>
------------	----------------

<b>D4.0</b>	MUT 402H
	MUT 402N


GH = 2mm

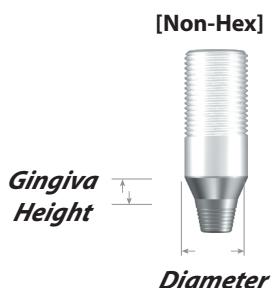**UCLA Abutment R/W/U**

<i>Hex</i>	<i>Non-Hex</i>
------------	----------------

<b>D4.5</b>	SUT 452H
	SUT 452N

GH = 2mm

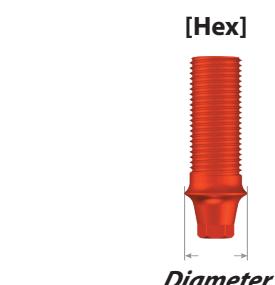**Plastic Cylinder mini**

<i>Hex</i>	<i>Non-Hex</i>
------------	----------------

<b>D4.0</b>	M-PSC 40H
	M-PSC 40N



**Plastic Cylinder R/W/U**

<i>Hex</i>	<i>Non-Hex</i>
------------	----------------


<b>D4.5</b>	S-PSC 45H
	S-PSC 45N

---

## Multi-Unit System

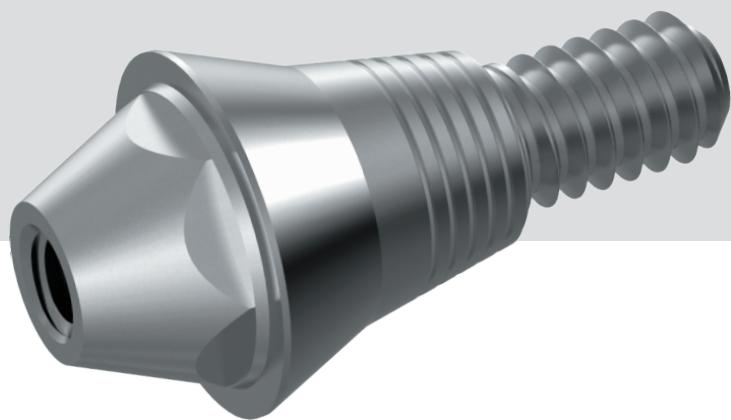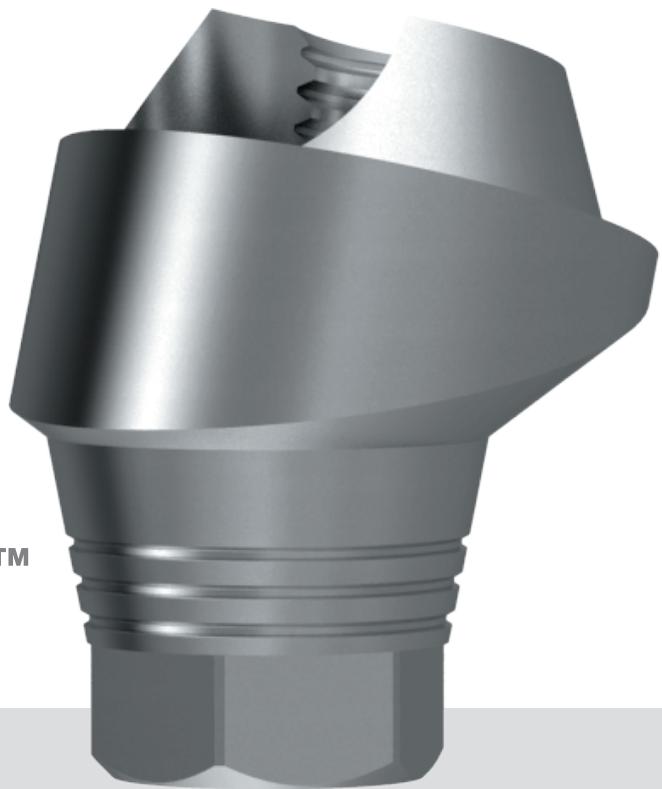

# Multi-Unit System Flow Chart

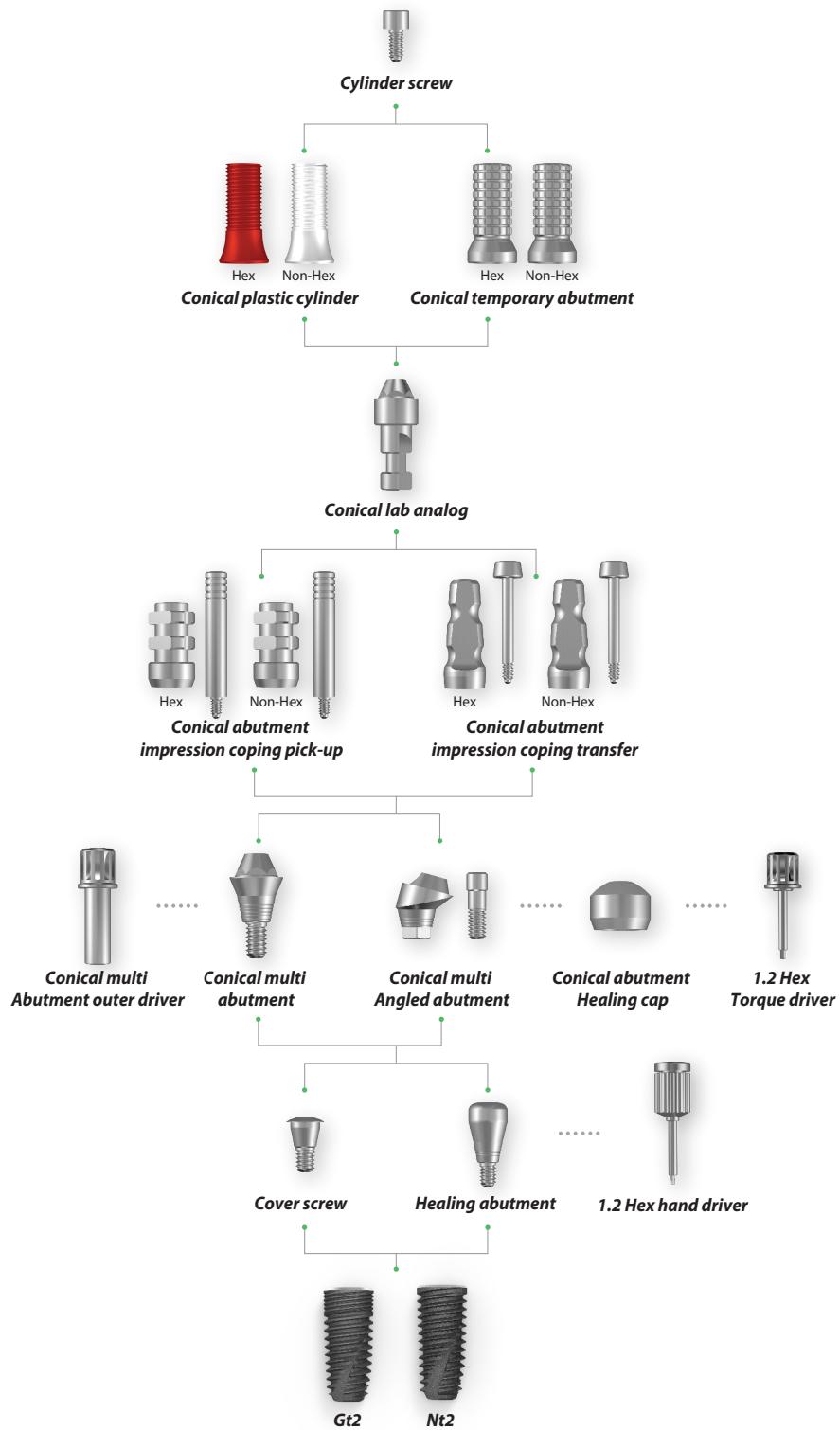

# Multi-Unit Submerged Abutment

- Mini
- Regular
- Wide
- Ultra-wide



## Conical multi abutment mini




	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.8</b>	MCMA 4810	MCMA 4820	MCMA 4830	MCMA 4840	MCMA 4850

## Conical multi abutment R/W/U



	<b>GH 1</b>	<b>GH 2</b>	<b>GH 3</b>	<b>GH 4</b>	<b>GH 5</b>
<b>D4.8</b>	CMA 4810	CMA 4820	CMA 4830	CMA 4840	CMA 4850

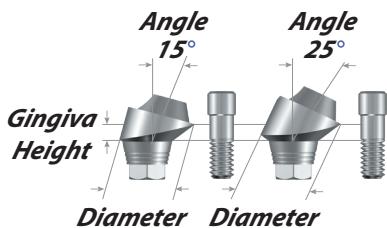

## Conical multi angled abutment mini




	<b>A</b>	<b>GH 1.5</b>	<b>GH 2.5</b>
<b>D4.8</b>	<b>15</b>	MCMAA 481515 H	MCMAA 482515 H
	<b>25</b>	MCMAA 481525 H	MCMAA 482525 H

## Conical multi angled abutment R/W/U



	<b>A</b>	<b>GH 1.5</b>	<b>GH 2.5</b>
<b>D4.8</b>	<b>15</b>	CMAA 481515 H	CMAA 482515 H
	<b>25</b>	CMAA 481525 H	CMAA 482525 H

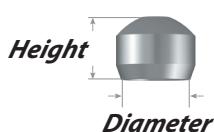

## Conical abutment healing cap M/R/W/U

**H6.0**

**D4.8** CAHC 600



## Conical multi abutment outer driver

**Short (8mm)**


**Long (15mm)**

**Hex 3.3** RHV 33 S

RHV 33 L



**Conical lab analog** M/R/W/U
**D4.8**

CLA 48

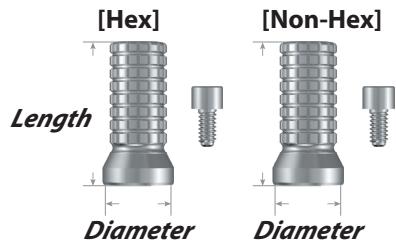
**Conical temporary abutment** M/R/W/U
**Length 12****Hex****Non-Hex****D4.8**


CTPA 48 H

CTPA 48 N

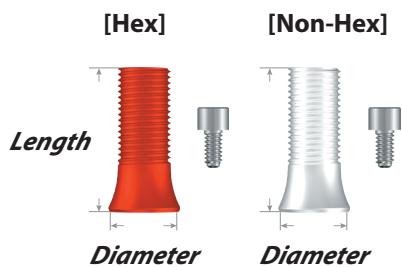
**Conical plastic cylinder** M/R/W/U
**Length 12****Hex****Non-Hex****D4.8**

CPSC 48 H

CPSC 48 N

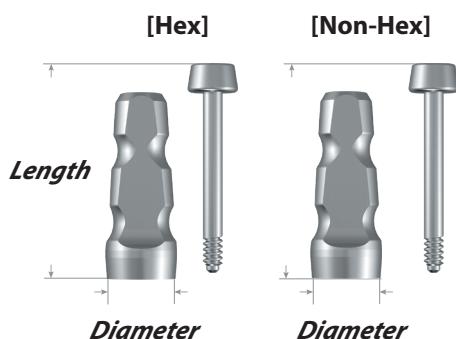

**Conical abutment  
impression coping (Transfer)** M/R/W/U
**Length 11****Length 15****Hex****Non-Hex****Hex****Non-Hex****D4.8**

CAICT 4811 H

CAICT 4811 N

CAICT 4815 H

CAICT 4815 N

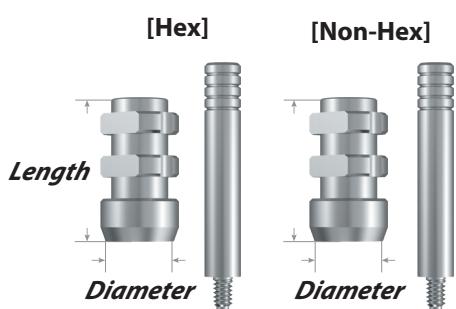
**Conical abutment  
Impression coping (Pick up)** M/R/W/U
**Length 10****Length 15****Hex****Non-Hex****Hex****Non-Hex****D4.8**


CAICP 4810 H

CAICP 4810 N

CAICP 4815 H

CAICP 4815 N

# Common Components of Surgical Kit

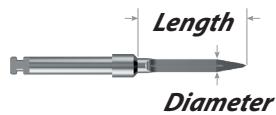

## Guide drill

	Diameter	Length
<b>GDR 20B</b>	2.0	15


- Easily forms the first hole in the initial drilling
- Marks the direction of the initial drilling in the cortical bone structure
- Only the triangular tip of the drill bit is used
- Bone density is assessed through the guide drill



## Drill extention

### **DRE 002**

- To extend the length of the used drills and other surgical equipment handpieces.

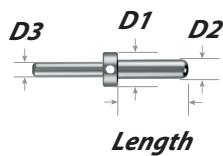

## Parallel pin

	<b>D1</b>	<b>D2</b>	<b>D3</b>	<b>L</b>
<b>TPP 50</b>	5.0	2.8	2.2	10

- Confirms the direction and distance in bone preparation.
- Confirms the distance of spaces in multi-insertions.



## Torque wrench

### **TRW 400 B**

- Used when inserting the fixture and fastening the screw
- Possible 15/25/35N tool adjustment



### Fixture driver Hex mini

	Hex
<i>For Handpiece</i>	<i>MMHL 002S</i> 2.1
	<i>MMHL 002L</i> 2.1
<i>For Torque wrench</i>	<i>RMHL 002S</i> 2.1
	<i>RMHL 002L</i> 2.1



### Fixture driver Hex R/W/U

	Hex
<i>For Handpiece</i>	<i>MHL 002S</i> 2.5
	<i>MHL 002L</i> 2.5
<i>For Torque wrench</i>	<i>RHL 002S</i> 2.5
	<i>RHL 002L</i> 2.5



### Fixture driver Octa R/W/U

	Octa
<i>For Handpiece</i>	<i>MOL 002S</i> 3.1
	<i>MOL 002L</i> 3.1
<i>For Torque wrench</i>	<i>ROL 002S</i> 3.1
	<i>ROL 002L</i> 3.1

#### For Handpiece

- Fastened with handpiece engine
- For Handpiece is used to insert and fasten the fixture
- Designed to prevent dropping when picking up the fixture to be fastened

#### For Torque wrench

- Fastened with torque wrench
- For Torque Wrench is used to insert and fasten the fixture
- Designed to prevent dropping when picking up the fixture to be fastened.

### 1.2 Hex driver

	Length	Hex
<i>Hand driver</i>	<i>THV 12SB</i> 8	1.2
	<i>THV 12LB</i> 15	1.2
<i>Machine driver</i>	<i>MHV 12SB</i> 8	1.2
	<i>MHV 12LB</i> 15	1.2
<i>Torque driver</i>	<i>RHV 12SB</i> 8	1.2
	<i>RHV 12LB</i> 15	1.2

#### Hand driver



- Hand driver is used when manually fastening the fixture with the joined abutment and screw

#### Machine driver


- Driver for engine

#### Torque driver

- Driver for fastening torque wrench

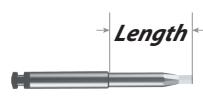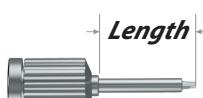

# Submerged full stop drill surgical kit

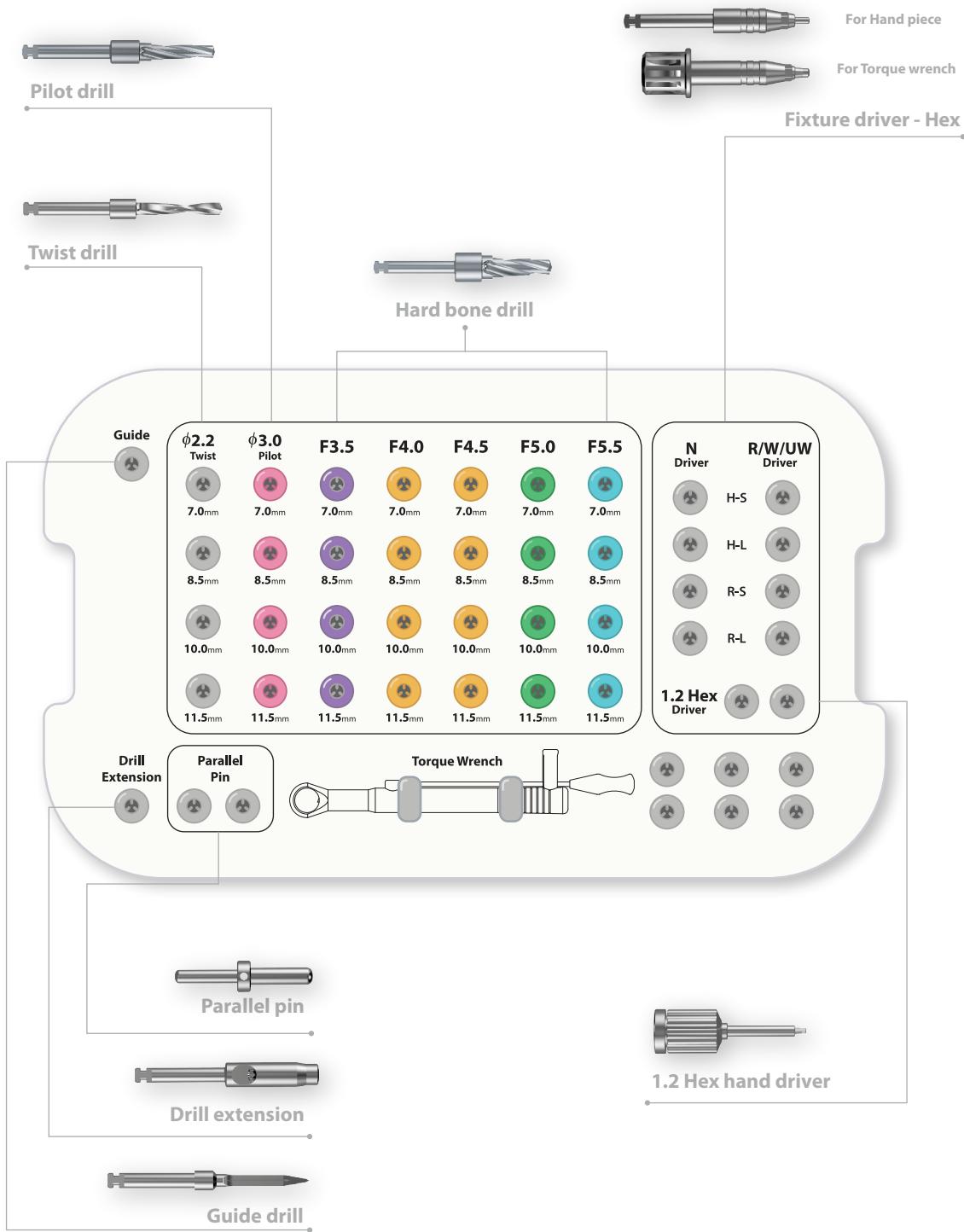

# Submerged Surgical drills



## Twist drill

	Diameter	Length
<b>TDR 22075</b>	2.2	7.5
<b>TDR 22085</b>	2.2	8.5
<b>TDR 2210</b>	2.2	10
<b>TDR 22115</b>	2.2	11.5
<b>TDR 2213</b>	2.2	13
<b>TDR 2215</b>	2.2	15

- Initial hole is formed at the marked region by the guide drill
- Caution is used to the adjacent space's depth and parallel



## Pilot drill

	D1	D2
<b>PDR 2230</b>	2.2	3.0

- After the initial drilling the Ø2.2 entry way is expanded to Ø3.0 for the tubal drill entry of both the tapered drill and straight drill

### Hard Bone drill




	<i>D1</i>	<i>D2</i>	<i>Length</i>
<b><i>IPD 35075</i></b>	2.6	3.3	7.5
<b><i>IPD 35085</i></b>	2.6	3.3	8.5
<b><i>IPD 3510</i></b>	2.6	3.3	10
<b><i>IPD 35115</i></b>	2.6	3.3	11.5
<b><i>IPD 3513</i></b>	2.6	3.3	13
<b><i>IPD 3515</i></b>	2.6	3.3	15
<b><i>IPD 40075</i></b>	3.1	3.8	7.5
<b><i>IPD 40085</i></b>	3.1	3.8	8.5
<b><i>IPD 4010</i></b>	3.1	3.8	10
<b><i>IPD 40115</i></b>	3.1	3.8	11.5
<b><i>IPD 4013</i></b>	3.1	3.8	13
<b><i>IPD 4015</i></b>	3.1	3.8	15
<b><i>IPD 45075</i></b>	3.6	4.3	7.5
<b><i>IPD 45085</i></b>	3.6	4.3	8.5
<b><i>IPD 4510</i></b>	3.6	4.3	10
<b><i>IPD 45115</i></b>	3.6	4.3	11.5
<b><i>IPD 4513</i></b>	3.6	4.3	13
<b><i>IPD 4515</i></b>	3.6	4.3	15
<b><i>IPD 50075</i></b>	4.1	4.8	7.5
<b><i>IPD 50085</i></b>	4.1	4.8	8.5
<b><i>IPD 5010</i></b>	4.1	4.8	10
<b><i>IPD 50115</i></b>	4.1	4.8	11.5
<b><i>IPD 5013</i></b>	4.1	4.8	13
<b><i>IPD 5015</i></b>	4.1	4.8	15
<b><i>IPD 55075</i></b>	4.6	5.3	7.5
<b><i>IPD 55085</i></b>	4.6	5.3	8.5
<b><i>IPD 5510</i></b>	4.6	5.3	10
<b><i>IPD 55115</i></b>	4.6	5.3	11.5
<b><i>IPD 5513</i></b>	4.6	5.3	13
<b><i>IPD 5515</i></b>	4.6	5.3	15
<b><i>IPD 60075</i></b>	5.1	5.8	7.5
<b><i>IPD 60085</i></b>	5.1	5.8	8.5
<b><i>IPD 6010</i></b>	5.1	5.8	10
<b><i>IPD 60115</i></b>	5.1	5.8	11.5
<b><i>IPD 6013</i></b>	5.1	5.8	13
<b><i>IPD 6015</i></b>	5.1	5.8	15

- Used to expand the dimension of the equivalent body size of the fixture to be inserted into the Ø2.2 hole that is formed by twist drilling.

- To minimize bone resistance in order to prevent bone crack, necrosis and others, drills are used in stages starting with the smallest diameter.

- Fixture's own body shape is almost equivalent to the body shape

# Drilling sequence

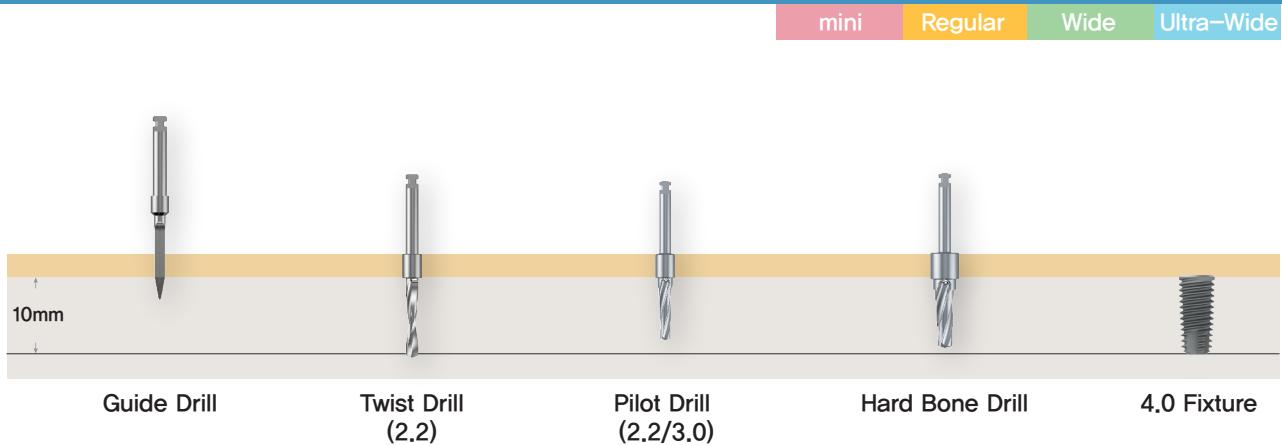

## 3.5 Fixture



H.D : Hard Bone Drill [D – 0.2]

## 4.0 Fixture



## 4.5 Fixture



## 5.0 Fixture



## 5.5 Fixture



## 6.0 Fixture



# **Ufit<sup>®</sup> dental implant**

---

Copyright © 2022 by T.Strong INC. All Right Reserved.





ISO 13485

[www.ufitimplant.com](http://www.ufitimplant.com)

Sales

## UFIT IMPLANT CO.,LTD

Tel. +82 1661 2832 Fax. +82 70 7543 4007  
#3102, 126, Beolmal-ro, Dongan-gu, Anyang-si,  
Gyeonggi-do, 14057, Rep. of KOREA

